

1 Check before use



The Lockcon Kit Klic100 is packed and sealed in a cardboard box.

All the components such as packaging, bottles and caps show the same unique number.

Select a kit you would like to proceed with.



- 1 check the box and make sure that the perforation is untouched and complete
- 2 check that all the kit components have the same unique ID-Number

2 Proceed

Use and proceed according to the WADA ISTI guidelines



- 1 Open the box by tearing off the perforation on the front side of the box.



- 2 Take the bottles out.
avoid touching the inside




- 3 Take the caps out.
avoid touching the inside



- 4 Pour the requested amount of urine into each bottle.
*30ml lower end of label → B sample
60ml upper end of label → A-sample*




- 5 Take the red cap and close the red-labeled bottle by turning clockwise. Proceed with the blue bottle the same way.

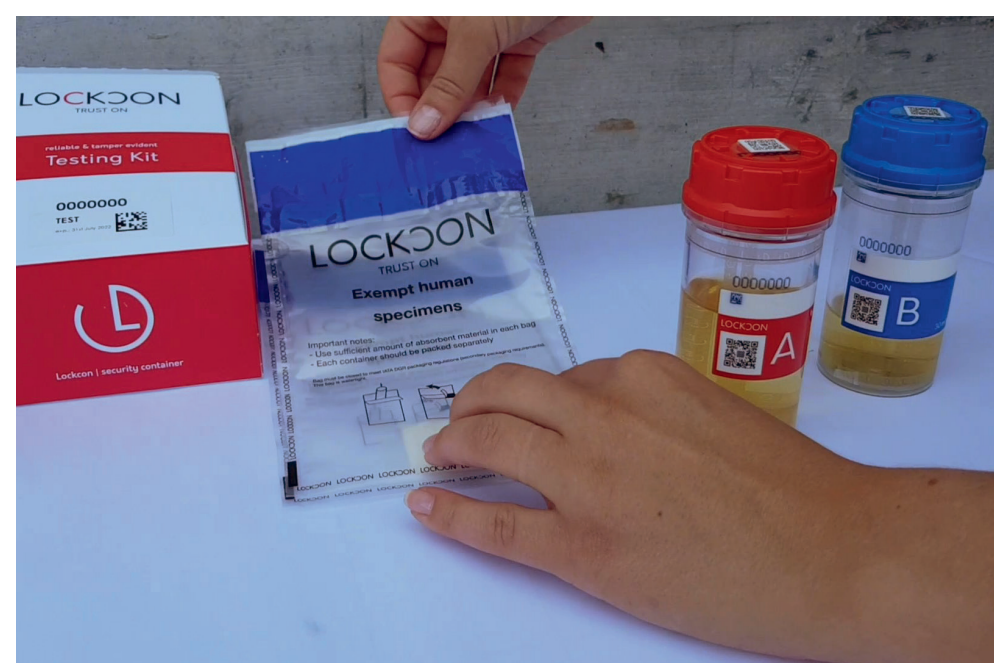

- 6 Take a watertight bag for each bottle and place it in there.




- 7 Tear off the white liner and press it firmly.

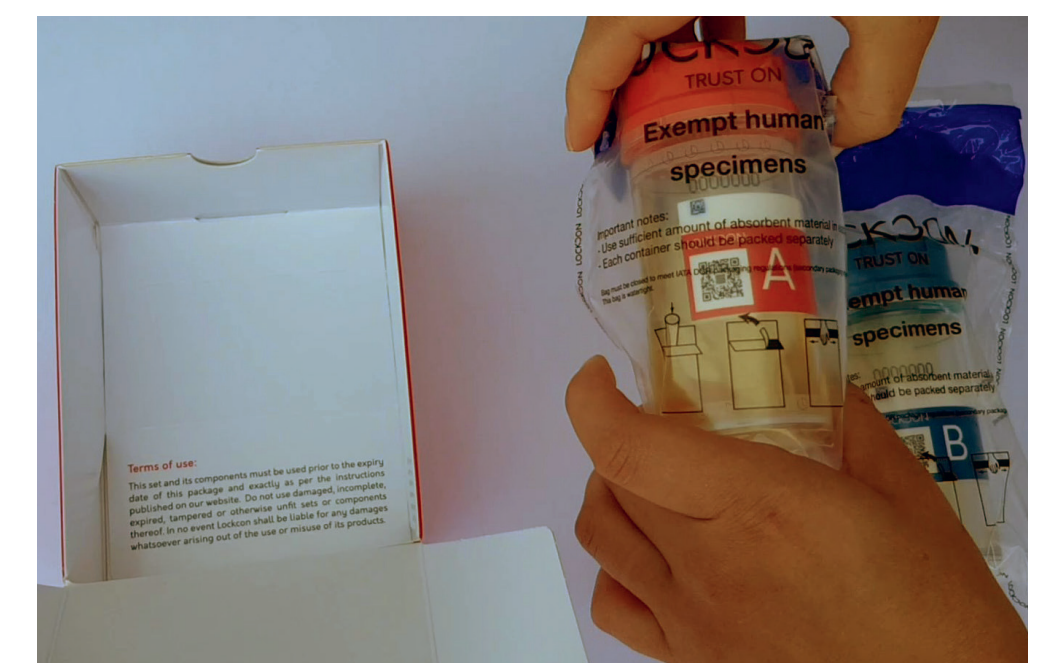

- 8 Place the filled bottles back into the cardboard box. To close the box, lead the upper front-end underneath the front side of the box.

3 Partial sample

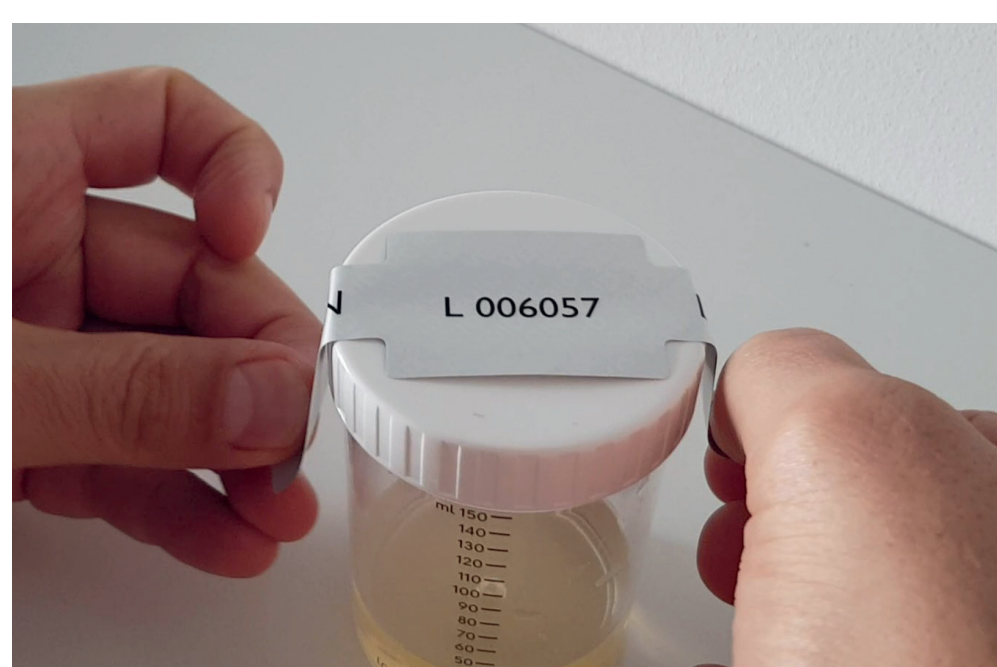

If you cannot provide enough urine (90 ml) at one time, you need to intermediately seal the collected urine. Take a uniquely numbered security void tape and place it over the closed screw cap of the urine collection vessel.

